## IFU

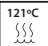
(Instruction for use)

### Prolongador de Brocas e Chaves / Drill and Drivers Extender / Prolongador de Fresas Y Llaves

PORTUGUÊS

ENGLISH

ESPAÑOL

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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to Emfils instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

## DESCRIPTION

The drill and driver extender (PB-2905) has a cylindrical shape that has a maximum diameter of 4.5 mm. At its upper tip, it has a counter-angle fitting and at its lower tip, it has a quick coupling for drills or counter-angle drivers, where the attachment is by contact with the flat surface of the extension. There is also a ring that engages internally in the extender, which supports the long axis of the drill or the driver, preventing it from being disconnected. The drill and driver extender is an integral part of the Emfils surgical kit.

The raw material used in all of the drivers is tempered stainless steel, according to ASTM F899.

## APPLICATION

The extender is a surgical instrument designed to increase the active reach of the drill or driver when the adjacent interdental height is greater than the useful height of the drill or driver.

## CONTRAINDICATIONS

The drill and driver extender do not present contraindications, provided they are used correctly for the purposes indicated.

## HANDLING

Once sterilized, the drill and driver extender should be used in compliance with all biosafety and surgical asepsis requirements.

All procedures must be performed by qualified professionals and the handling of the product is inherent in the training of the professional who will be responsible for using the instruments in the best possible way, respecting the Emfils indications for each instrument.

Attach the extender to the counter-angle, and the drill or driver to the extender. Place it at the specified rotation, according to the instructions for use of the selected implant or component. Take the assembled extender to the location of the drill or snap it into the implant or component. Start the counter-angle engine and do the specific work required. When replacing the drill or driver, remove the former by means of circular movements so as not to compromise the positioning of the internal ring of the extension. In case of use of the extender in a drilling, the pressure must not be excessive and abundant irrigation, manual or combined, is required using the counter-angle pump.

Important points in a bone drilling:

- To minimize surgical trauma, which is critical to the success of osseointegration of the implant, the sequence of drills must be respected, as well as irrigation with saline for bone cooling. See the drill sequencing in the updated Emfils catalog or at [www.emfils.com](http://www.emfils.com).
- Avoid interrupting motor rotation with the drill inside the surgical cavity, this may make a difficult to remove the drill or cause the drill break.
- Always the next drill in the sequence will use the previous drill as a guide and follow the same course, so a good drilling of the initial drill, as defined in the planning, is of fundamental importance so that no deviations occur, nor is there a need of corrections, taking advantage in the best possible way the amount of bone existing.
- Inadequate planning may compromise the performance of the implant / prosthesis assembly, resulting in system failure, such as loss or fracture of the implant, thread loosening or fracture of the prosthetic screw;
- Make sure you have all the necessary instruments for the surgery according to surgical planning;
- Before each procedure, check the conditions of the Emfils instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear;
- Before each procedure, make sure that the extender is fully engaged in the counter-angle and in the drill or the driver, otherwise deformation, shearing or even breaking of the drill, driver, implant or component may occur in the torque;
- The use of excessive force can compromise the surgical act and damage the key, implant or component;
- When replacing the drill or driver, remove the previous one by means of circular movements so as not to compromise the positioning of the internal ring of the extension;
- Always use the sequence of Emfils products. The use of instruments and / or components from other manufacturers and systems does not ensure the perfect function and exempts Emfils from offering any product warranty.

## SANITATION AND STERILIZATION

The extender is reusable, provided non-sterile and should be sanitized and sterilized before and after use by following the steps mentioned in the link below:

### Novo Colosso's Surgical Kit Cleaning Manual

## PRESENTATION

The extender is individually packaged in tyvek® surgical paper and transparent film display. The instrument undergoes a decontamination process that eliminates any type of dirtiness, but the sanitation process, described above, must be followed as soon as it's first used.

## PRECAUTIONS

- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use steam and pressure autoclave;
- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Due to the reduced opening of the mouth in the posterior region, the use of the instrument may be difficult;
- Ensure that instruments are not swallowed or aspirated by the person receiving the product (patient);
- Inadequate planning may compromise the performance of the implant / prosthesis assembly, resulting in system failure, such as loss or fracture of the implant, thread loosening or fracture of the prosthetic screw;
- Make sure you have all the necessary instruments for the surgery according to surgical planning;
- Before each procedure, check the conditions of the Emfils instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear;
- Before each procedure, make sure that the extender is fully engaged in the counter-angle and in the drill or the driver, otherwise deformation, shearing or even breaking of the drill, driver, implant or component may occur in the torque;
- The use of excessive force can compromise the surgical act and damage the key, implant or component;
- When replacing the drill or driver, remove the previous one by means of circular movements so as not to compromise the positioning of the internal ring of the extension;
- Always use the sequence of Emfils products. The use of instruments and / or components from other manufacturers and systems does not ensure the perfect function and exempts Emfils from offering any product warranty.

## ADVERSE EFFECTS

Adverse effects will only occur if the choice of instruments is inadequate, which can lead to an unsuccessful drilling or damage to the implant or component to be installed.

## STORAGE CONDITIONS

The product should be stored in a dry place and in the original packaging at room temperature and not expose to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

## DISPOSAL OF MATERIAL

The professional must always evaluate the conditions of the instruments after each use. If the instrument is detected to be damaged, markings erased, deformed or excessive wear, must be disposed of in accordance with current regulations, complying with the environmental requirements and the criterion of potentially hazardous products, taking into account the different levels of contamination, physical risks and selective collection.

## LIFE CYCLE

The extender should be disposed when it loses its functionality.

## SHELF LIFE

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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