## IFU

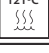
(Instruction for use)

### Chave de Instalação Guiada / Installation Driver Guide / Llave de Instalación Guiada

PORTUGUÊS

ENGLISH

ESPAÑOL

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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to Emfils instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

## DESCRIPTION

The Guide installation keys are subdivided into 4 groups: Short / Narrow, Short / Regular, Long / Narrow and Long / Regular and have 3 prosthetic connection models: hexagonal 2.0 mm, hexagonal 2.1 mm and hexagonal 2,4 mm. In the socket of the top that connects to the gear or to the contra angle, all keys have counter-angle locking. In total, we have 12 key options only for use in the In fi ls Implant System through the surgical technique Guide. Hexagonal wrenches for implant installation In fi ls (2.0, 2.1 and 2.4 mm), the lower end has a collection system that secures the implant to the key and helps to remove the transparent tube from the package without any manipulation of the implant is necessary.

The Guide installation keys are guided through the regular or narrow washers, according to the surgical planning of the Guide technique. In narrow interdental spaces, the narrow washer can be used to prevent collision between the washers, therefore the narrow wrench should be used.

The laser markings take into account the actual implant length, with an additional 11 mm length to compensate for the soft tissue thickness and height of the washer and the Drill Guide. The laser engraving with the size of its slot serves to facilitate its identification and storage by the user.

The raw material used in all keys is stainless steel tempered steel, according to ASTM F899.

All key models for Guide installation are presented in the table below:

Sales Code	Connection Dimension						
	Implant			Connection	Length	Sleeves diameter	Fit
	CM	HE	HI				
CECG-2020	X			Hex. 2,0	Short	Narrow	Counter-Angle
CRCG-2020	X			Hex. 2,0	Short	Regular	Counter-Angle
CELG-2020	X			Hex. 2,0	Long	Narrow	Counter-Angle
CRLG-2020	X			Hex. 2,0	Long	Regular	Counter-Angle
CECG-2121		X		Hex. 2,1	Short	Narrow	Counter-Angle
CRCG-2121		X		Hex. 2,1	Short	Regular	Counter-Angle
CELG-2121		X		Hex. 2,1	Long	Narrow	Counter-Angle
CRLG-2121		X		Hex. 2,1	Long	Regular	Counter-Angle
CECG-2424			X	Hex. 2,4	Short	Narrow	Counter-Angle
CRCG-2424			X	Hex. 2,4	Short	Regular	Counter-Angle
CELG-2424			X	Hex. 2,4	Long	Narrow	Counter-Angle
CRLG-2424			X	Hex. 2,4	Long	Regular	Counter-Angle

## APPLICATION

The installation keys in the Guide tabs are indicated for installation of implants using the surgical technique Guide as shown in the table above. Short Guide installation keys are only suitable for installation of 7.0mm, 8.5mm and 10.0mm long implants. Long wrenches are indicated for 7.0mm, 8.5mm, 10.0mm, 11.5mm, 13.0mm, 14.5mm and 16mm long implants.

## CONTRAINDICATIONS

The Guide installation keys do not have contraindications, provided they are used correctly for the indicated purposes.

## HANDLING

Once sterilized, the Guide installation key should be handled only in a sterile environment, by properly trained professionals, and in appropriate attire at the time of implant surgery.

All procedures must be carried out by qualified professionals and the handling of the installation key is inherent to the training of the professional, who will be responsible for using the instruments in the best possible way, respecting the indications of the Emphasis for each instrument.

To install the respective implant in the surgical socket check and choose the template according to the fixed washer in the surgical guide. Insert the key into the contra-angle motor or the Torquimetro and insert the key into the washer. With hourly movements, install the implant up to the planned height based on the recording references or until the end of the stop. It is important to respect the maximum torque of the recommended implant. For removal or uninstallation, use the same locking system, but the turn of the key must be counterclockwise.

Important points in using the keys:

- Avoid side load.
- The keys do not have torque control.
- Use of excessive force may damage the product.
- check the washer match.

## SANITATION AND STERILIZATION

Guide installation keys are reusable, provided non-sterile and must be sanitized and sterilized before and after use, following the recommendations in the document available at the link below:

[Novo Colosso's Surgical Kit Cleaning Manual](#)

## PRESENTATION

The Emfils installation drivers are individually packaged in tyvek® surgical paper and transparent film display. The instrument undergoes a decontamination process that eliminates any type of dirtiness, but the sanitation process, described above, must be followed as soon as its first used.

## PRECAUTIONS

- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use steam and pressure autoclave;
- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Check the correspondence of the prosthetic connection between the driver and the implant, or component that you want to install;
- Avoid side load on the switch;
- The drivers have no torque control. Only when used through the torque wrench will any measurement be possible;
- Maximum care is recommended when using the instruments so that they are not swallowed or aspirated by the patient;
- Before each procedure, check the conditions of the Emfils instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear.
- Before each procedure, make sure that the driver is properly fitted of the driver into the torque wrench gear or counter-angle head and the implant or component, otherwise it may occur the deformation, shearing, or even breakage of the driver, implant or component may occur in the application of torque;

- The use of excessive force can compromise the surgical act and damage the key, implant or component;

Always use the sequence of Emfils products. The use of instruments and / or prosthetic components from other manufacturers and systems does not ensure the perfect function and exempts Emfils from offering any product warranty. The drivers for installing components from other systems are indicated for use in other implant systems as recommended by the component manufacturer.

#### ADVERSE EFFECTS

Adverse effects will only occur if the choice of instrument or its handling is inadequate, which can cause damage to the surgical sequence of the implant to be installed.

#### STORAGE CONDITIONS

The product should be stored in a dry place and in the original packaging at room temperature and not expose to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

#### DISPOSAL OF MATERIAL

The operator must always evaluate the condition of the instrumentation after each use and it must be discarded in accordance with current regulations, complying with the environmental requirements and the criterion of potentially dangerous products, and if the instrument is damaged, markings erased, deformed and worn. , taking into account the different levels of contamination, physical risks and selective collection

#### LIFE CYCLE

Emfils installation keys should be discarded when they lose their functionality or the user detects excessive wear on the dock.

#### SHELF LIFE

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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