IFU

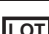
(Instruction for use)

Estojo / Case / Estuche

PORTUGUÊS

ENGLISH

ESPAÑOL

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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so for best results it should only be used with original associated components according to Emfils instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthetists and Oral Maxillofacial Surgeons.

DESCRIPTION

Emfils plastic cases are trays for the storage of surgical and / or prosthetic instruments. Emfils plastic cases are autoclavable up to 135 ° C. They are made of special polymer and are manufactured in accordance with ISO 10993. The packaging supports are manufactured in non-toxic silicone. It has laser markings that guide the user during procedures and in storage. The assembly is protected by a translucent cap identified.

The Emfils plastic cases are presented in the table below:

| Sales Code | Application | Size | | |
|-------------|---|--------|-------|--------|
| | | length | width | height |
| ESK-1813 | Surgical Kit Case | 180 | 140 | 70 |
| EKG-6290 | Kit of the Emfils Guide Kit | 180 | 140 | 70 |
| EEKO-5590 | Kit for Osteotomes | 195 | 90 | 55 |
| EEKSD-5590 | Kit stop kit | 195 | 90 | 35 |
| EEKP-4490 | Kit for Prosthetic Kit | 195 | 90 | 44 |
| EKC-2807A9 | Case for Bone Compactor Kit | 81 | 67 | 45 |
| EKE-2807A9 | Bone Expander Kit Case | 81 | 67 | 45 |
| EKEO-2807A9 | Case for Bone Expander Kit with Thread | 81 | 67 | 45 |
| KANC-8045 | Kit for Adaptation Kit of the New Emfils System | 81 | 67 | 45 |

APPLICATION

The plastic cases are indicated to support and securely secure the surgical and / or prosthetic instruments during autoclave use and sterilization. The use of this product rationalizes the storage and organization of the instruments during and after the dental procedure. Each case has silicone supports for the fixation of the necessary instruments for the specific procedure intended.

CONTRAINDICATIONS

Plastic cases have no contraindications, provided they are used correctly for the purposes indicated.

HANDLING

For large tray cases, remove the translucent cover to access the instrumentation by pressing the pressure tabs on the longitudinal edges of the case. To close the case, position the cover on the base and press, the fastening will take place automatically.

For large, hinged cases, press the center latch and lift the lid in a tilting motion. To close the case, tilt the cover until it locks.

For small cases, simply lift the lid in a tilting motion.

For sterilization, the cleaned, cleaned and dried instrumentation must be packed in the case and it must be wrapped in surgical grade autoclave paper. During sterilization, do not touch the case on the autoclave walls.

SANITATION AND STERILIZATION

Plastic cases are reusable, supplied non-sterile and must be sanitized and sterilized before and after use, following the recommendations of the document available at the link below:

PRESENTATION

Emfils plastic cases are individually wrapped and secured by unique seals in both the length and width of the case. The cases undergo a cleaning process that eliminates any kind of dirt, but the hygiene process, described above, must be followed as soon as it is first used.

PRECAUTIONS

- Do not use if the packaging has been tampered

with;

- This product should be used sterile. Do not sterilize under dry heat (greenhouse), always use steam autoclave and pressure;
- Chemical sterilization is not recommended;
- Do not autoclave the case with the seals;
- Do not touch the case on the autoclave walls to avoid melting of the case due to excess temperature;
- Place the autoclave water meter recommended by the manufacturer. The lack of water during the sterilization cycle can compromise the efficacy of the autoclave, lead to melting of the case, as well as damage of the instruments;
- Case contact with acrylic resin fluid results in damage to the polymer structure of the case;
- For sterilization, instruments should be individually sanitized, as directed above, and packaged in the case, which must be wrapped in surgical grade autoclave paper;
- Make sure you have all the necessary instruments to perform the surgery according to surgical planning;
- Before each procedure check the conditions of the Emfils instruments, always respecting their life. Replace instruments in case of damage, missing markings, deformation and wear;
- Always use the sequence of Emfils products. The use of instrumentation and / or prosthetic components from other manufacturers does not guarantee the perfect function of the Emfils implant system and disclaims any warranty of the product.

ADVERSE EFFECTS

Adverse effects will only occur if the recommendations for use described are not adhered to.

STORAGE CONDITIONS

The product should be stored in a dry place in its original packaging at room temperature and not exposed to direct sunlight. Incorrect storage can influence the original characteristics of the packaging and the product.

DISPOSAL OF MATERIAL

The professional must always evaluate the conditions of the instruments after each use, and if it is detected that the instrument is damaged, markings erased, deformations and wear, they should be discarded according to the current norms, obeying the environmental requirements and the criterion of potentially dangerous products, taking into account the different levels of contamination, physical risks and selective collection.

LIFE CYCLE

Emfils plastic cases should be discarded when they lose their functionality or the user detects excessive wear.

SHELF LIFE

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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