IFU

(Instruction for use)

Brocas / Drills / Fresas

PORTUGUÊS

ENGLISH

ESPAÑOL

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- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Should be careful in the drill sanitation process because it is a sharp metal instrument, avoiding any type of physical trauma;
- Be sure to use the drill compatible with the specified drill sequence according to the dimensions of the planned implant;
- Surgical drills should not be re-sharped;
- The non-replacement of a drill, as recommended by Medens, can generate undue bone heating, compromising the success of the procedure. The drills should not be with their sharpening committed. Refer to the "Life Cycle" section at the end of this instruction manual;
- Due to the reduced opening of the mouth in the posterior region, the use of the instrument may be difficult;
- Before each procedure, check the conditions of the Medens instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear.
- Before each procedure, make sure that the drill is perfectly coupled into the counter-angle, otherwise deformation or even breaking of the drill may occur;
- The use of excessive force can compromise the surgical act;
- Always use the sequence of Medens products. The use of instruments and / or components from other manufacturers and systems does not ensure the perfect function and exempts Medens from offering any product warranty.

ADVERSE EFFECTS

Adverse effects will only occur if the choice of instrument or its handling is inadequate, which can lead to unsuccessful of the drilling and even damage to the patient.

STORAGE CONDITIONS

The product should be stored in a dry place and in the original packaging at room temperature and not expose to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

DISPOSAL OF MATERIAL

The professional must always evaluate the conditions of the instruments after each use. If the instrument is detected to be damaged, markings erased, deformed or excessive wear, must be disposed of in accordance with current regulations, complying with the environmental requirements and the criterion of potentially hazardous products, taking into account the different levels of contamination, physical risks and selective collection.

LIFE CYCLE

This product is recommended for up to 30 uses, provided that Medens-recommended conditions of use are followed. Regardless of the number of times the instrument has been used, the professional must always evaluate the instrument's conditions after each use.

SHELF LIFE

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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