IFU


(Instruction for use)

Chave de Instalação / Installation Driver / Llave de Instalación

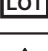PORTUGUÊS

ENGLISH

ESPAÑOL

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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to Medens instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

DESCRIPTION

The Medens installation keys are subdivided into 3 groups: Medens implant installation key, Medens prosthetic component installation key and Prosthetic components (other systems) installation key.

The keys for implant installation Medens have 4 models: hexagonal 2.0 mm, hexagonal 2.1 mm, hexagonal 2.4 mm, and square 2.0 mm in short, medium and long sizes. In the top fitting, which connects to the gear or contra angle, all short and long size keys have a 3.9 mm universal square fit and all medium braces have snap fit. In total, we have 12 key options for use only in the Medens Implant System. Hexagonal wrenches for implant installation Medens (2.0, 2.1 and 2.4 mm), the lower end has a collection system that secures the implant to the key and helps to remove the transparent tube from the package without any manipulation of the implant is necessary. There is also the option, if the user understands that he needs a longer key, he should use the key extension, which increases the actuation length of the medium angle locking switches, where another option is obtained for each model. four options. All these keys and options are an integral part of the surgical kit Medens.

The keys for installation of Medens prosthetic components are available in the models: 1.2mm hexagonal (short, medium and long), 2.0mm hexagonal (short and medium) and 3.0mm hexagonal (average). The short and long size keys have a 3.9 mm universal square fit and the medium keys have snap fit for angle. If necessary, the medium-sized keys can be extended using the key extension. Except the 3.0mm hexagonal wrench that makes up the prosthetic kit, the other keys and options are an integral part of the surgical kit Medens.

We also have the keys for the installation of prosthetic components of the market (other systems), where Em fi ls composed a prosthetic kit that contains practically all models of keys existing in the Brazilian market, in the medium size option, with counter angle fitting, length of operation increased with the use of the key extension. All keys for installation of components of other systems are an integral part of the prosthetic kit Medens.

All keys are laser engraved with the size of their slot for easy identification and storage by the user. All internal locking wrenches have a transverse hole in the side of the body for easy cleaning.

The raw material used in all keys is stainless steel tempered steel, according to ASTM F899.

All key models for installation are presented in the table below:

Sales Code	Installation Component							Screw	Connection Dimension	Connection Type		Lenght	Interface Type
	Implant						Internal			External			
	MT (CM)	ZL (LZ)	EH (HE)	IH (HI)	OBA (CUP)	OBO (CUO)							
IHC-1214							X	Hex. 1,2	X		Short	Sqr. 3,9	
IHC-2014	X							Hex. 2,0	X		Short	Sqr. 3,9	
IHC-2114			X					Hex. 2,1	X		Short	Sqr. 3,9	
IHC-2414				X				Hex. 2,4	X		Short	Sqr. 3,9	
ILCA-1226							X	Hex. 1,2	X		Long	Sqr. 3,9	
ILCA-2026	X							Hex. 2,0	X		Long	Sqr. 3,9	
ILCA-2126			X					Hex. 2,1	X		Long	Sqr. 3,9	
ILCA-2426				X				Hex. 2,4	X		Long	Sqr. 3,9	
IMCA-1221							X	Hex. 1,2	X		Medium	Counter Angle	
IMCA-2021	X							Hex. 2,0	X		Medium	Counter Angle	
IMCA-2121			X					Hex. 2,1	X		Medium	Counter Angle	
IMCA-2421				X				Hex. 2,4	X		Medium	Counter Angle	
IQC-2014					X	X		Sqr. 2,0	X		Short	Sqr. 3,9	
IQLCA-2026					X	X		Sqr. 2,0	X		Long	Counter Angle	
IQMCA-2021					X	X		Sqr. 2,0	X		Medium	Counter Angle	
IFC-2014					X	X		Hex. 2,0	X		Short	Sqr. 3,9	
IMCA-0921	---	---	---	---	---	---	---	Hex. 0,9	X		Medium	Counter Angle	
IMCA-1621	---	---	---	---	---	---	---	Hex. 1,6	X		Medium	Counter Angle	
IMFCA-2021	---	---	---	---	---	---	---	Hex. 2,0	X		Medium	Counter Angle	
IMFCA-2521	---	---	---	---	---	---	---	Hex. 2,5	X		Medium	Counter Angle	
IQMCA-1321	---	---	---	---	---	---	---	Sqr. 1,3	X		Medium	Counter Angle	
IQMCA-3021	---	---	---	---	---	---	---	Sqr. 3,0	X		Medium	Counter Angle	
IFMCA-0321	---	---	---	---	---	---	---	Slot 0,4	X		Medium	Counter Angle	
ITMCA-1721	---	---	---	---	---	---	---	Torx 1,6	X		Medium	Counter Angle	

A distinctive feature found in our 1.2 mm hexagonal installation keys is the Collection System, which enables the efficient fastening of the screw at the tip of the key, preventing screw drop, contamination, or even risks of the screw being aspirated or swallowed by the patient.

APPLICATION

The keys for Medens components are indicated for installation of implants and Medens system components as shown in the table. Keys for installing components from other systems should be used as recommended by the component manufacturer.

CONTRAINDICATIONS

The bi-digital drivers do not present contraindications, provided they are used correctly for the purposes indicated.

HANDLING

Once sterilized, the installation driver should only be handled in a sterile environment by properly trained professionals and in appropriate costume at the time of surgery for the installation of dental implants.

All procedures must be performed by qualified professionals and the handling of the product is inherent in the training of the professional who will be responsible for using the instruments in the best possible way, respecting the Medens indications for each instrument.

Check the correspondence of the prosthetic connection between the driver and the implant, or component that you want to install. Attach the driver interface to the torque wrench gear or counter-angle head, depending on the key chosen that best meets the user's need. Fit the tip of the driver to the implant or component and rotate clockwise until the planned torque is reached, always respecting the maximum torque recommended by Medens for each product. For removal or disassembly, use the same attachment system, but the turn of the driver must be counterclockwise.

Important points in using the drivers:

- Avoid side loading.
- The drivers do not have torque control.
- Excessive force use can damage the product.

SANITATION AND STERILIZATION

The Medens installation driver is reusable, provided non-sterile and should be sanitized and sterilized before and after use by following the steps mentioned in the link below:

[Surgical Kit Cleaning Manual](#)

PRESENTATION

The Medens installation drivers are individually packaged in tyvek® surgical paper and transparent film display. The instrument undergoes a decontamination process that eliminates any type of dirtiness, but the sanitation process, described above, must be followed as soon as its first used.

PRECAUTIONS

- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use steam and pressure autoclave;

- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Check the correspondence of the prosthetic connection between the driver and the implant, or component that you want to install;
- Avoid side load on the switch;
- The drivers have no torque control. Only when used through the torque wrench will any measurement be possible;
- Maximum care is recommended when using the instruments so that they are not swallowed or aspirated by the patient;
- Before each procedure, check the conditions of the Medens instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear.
- Before each procedure, make sure that the driver is properly fitted of the driver into the torque wrench gear or counter-angle head and the implant or component, otherwise it may occur the deformation, shearing, or even breakage of the driver, implant or component may occur in the application of torque;
- The use of excessive force can compromise the surgical act and damage the key, implant or component;

Please, contact the authorized distributor.

Always use the sequence of Medens products. The use of instruments and / or prosthetic components from other manufacturers and systems does not ensure the perfect function and exempts Medens from offering any product warranty. The drivers for installing components from other systems are indicated for use in other implant systems as recommended by the component manufacturer.

ADVERSE EFFECTS

Adverse effects will only occur if the choice of instrument or its handling is inadequate, which can cause damage to the surgical sequence of the implant to be installed.

STORAGE CONDITIONS

The product should be stored in a dry place and in the original packaging at room temperature and not expose to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

DISPOSAL OF MATERIAL

The professional must always evaluate the conditions of the instruments after each use. If the instrument is detected to be damaged, markings erased, deformed or excessive wear, must be disposed of in accordance with current regulations, complying with the environmental requirements and the criterion of potentially hazardous products, taking into account the different levels of contamination, physical risks and selective collection.

LIFE CYCLE

The Medens installation driver must be disposed when it loses its functionality, or the user detects excessive wear on the interface geometry.

SHELF LIFE

Written on the product label.

Not all products are available in all countries.

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