# IFU


(Instruction for use)

## Easy Drill

PORTUGUÊS

ENGLISH

ESPAÑOL

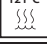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This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to Medens instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

**DESCRIPTION**

The Medens Easy Drills are a set of 32 pieces designed to mechanically limit the depth of the drilling, giving the user the confidence of not drill too much or too little for the planned implant installation. The Easy Drills are divided into 4 models, one for each two diameters of surgical drill. Each model has 8 lengths, one for each Medens implant length. The Easy Drills are tubular in shape and feature a snap-fit system that attaches to the drill body and abuts the center ring of larger diameter. All parts are identified by colored anodization and laser engraving with the diameter of the drill bit and the length of the implant to be used for ease of use by the user.

The raw material used is titanium alloy (grade 5), according to ASTM F136.

The Medens Easy Drills are presented in the table below:

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**APPLICATION**

The Easy Drills are instrumental used as accessories for surgical drills during the installation procedure of Medens dental implants, basically their function is to mechanically limit the depth of perforation of the bone tissue to the installation of the planned implant model.

**CONTRAINDICATIONS**

The Medens Easy Drills do not present contraindications, provided they are used correctly for the purposes indicated and used by a specialized professional, who is responsible for the adequate planning of the surgical procedure where the Easy Drill will be used.

**HANDLING**

Once sterilized, the Easy Drills should be handled only in a sterile environment, by properly trained professionals and in appropriate attire at the time of surgery to install dental implants.

All procedures must be performed by qualified professionals and the handling of the product is inherent in the training of the professional who will be responsible for using the instruments in the best possible way, respecting the Medens indications for each instrument.

Select the Easy Drill that meets the diameter of the drill and the length of the perforation you want to perform. Assemble the Easy Drill on the active body of the drill bit seating the part in the center ring of the drill bit. Verify if the length of the chosen Easy Drill matches the laser marking of the surgical drill. Install the drill into the counter-angle and select the specified rotation according the instructions for use of the selected implant. Take the drill bit to the surgical alveolus, as planned. Start the engine and drill until the Easy Drill outer body touches the surface of the bone and prevents the drill from drilling deeper. The perforation must be performed with complete insertion e removal movements of the drill. During the perforation, the pressure should not be excessive, and abundant irrigation, manual or combined, is required using the motor pump. Remove the Easy Drill by manually pulling the part out of the body of the surgical drill.

Important points in a bone drilling:

- To minimize surgical trauma, which is critical to the success of osseointegration of the implant, the sequence of drills must be respected, as well as irrigation with saline for bone cooling. See the drill sequencing in the updated Medens catalog or at [www.medens.com.br](http://www.medens.com.br).
- Avoid interrupting motor rotation with the drill inside the surgical cavity, this may make a difficult to remove the drill or cause the drill break.
- Always the next drill in the sequence will use the previous drill as a guide and follow the same course, so a good drilling of the initial drill, as defined in the planning, is of fundamental importance so that no deviations occur, nor is there a need of corrections, taking advantage, in the best possible way, the amount of bone existing.

**SANITATION**

The Medens Easy Drills are reusable, provided non-sterile and should be sanitized and sterilized before and after use by following the steps mentioned in the link below:

**PRESENTATION**

The Medens Easy Drills are individually packaged in tyvek® surgical paper and transparent film display. The instrument undergoes a decontamination process that eliminates any type of dirtiness, but the sanitation process, described above, must be followed as soon as it's first used.

**STERILIZATION**

Supplied non-sterile, being indicated for re-use, provided that the sanitation process be respected.


**PRECAUTIONS**

- Do not wash with the rough part of the sponge or abrasive materials, there may be removal of anodizing and / or engraving;
- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use steam and pressure autoclave;
- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Care must be taken in assembling the Easy Drill in the drill because it is a metal cutting tool, so as not to suffer any kind of physical injury;
- Be sure to use the drill and Easy Drill compatible with the indicated sequence, according to the dimensions of the planned implant;
- Due to the reduced opening of the mouth in the posterior region, the use of the instrument may be difficult. Attention also


to the available inter-occlusal space;

- Ensure that instruments are not swallowed or aspirated by the person receiving the product (patient);
- Inadequate planning may compromise the performance of the implant / prosthesis assembly, resulting in system failure, such as loss or fracture of the implant, thread loosening or fracture of the prosthetic screw;
- Make sure you have all the necessary instruments for the surgery according to surgical planning;
- Before each procedure, check the conditions of the Medens instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear;
- Before each procedure, make sure that the Easy Drill is properly engaged and seated in the surgical drill, otherwise drilling may not meet the planning requirements;
- The use of excessive force can compromise the surgical act;
- Always use the sequence of Medens products. The use of instruments and / or components from other manufacturers and systems does not ensure the perfect function and exempts Medens from offering any product warranty.

**KIT EASY DRILL**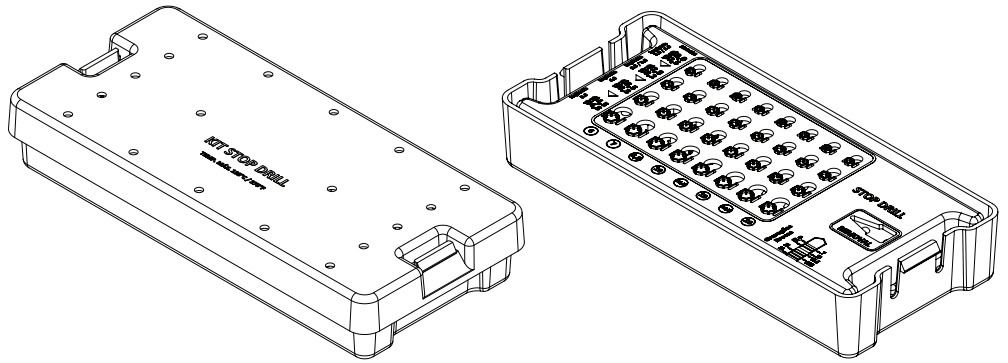

**EASY DRILLS**

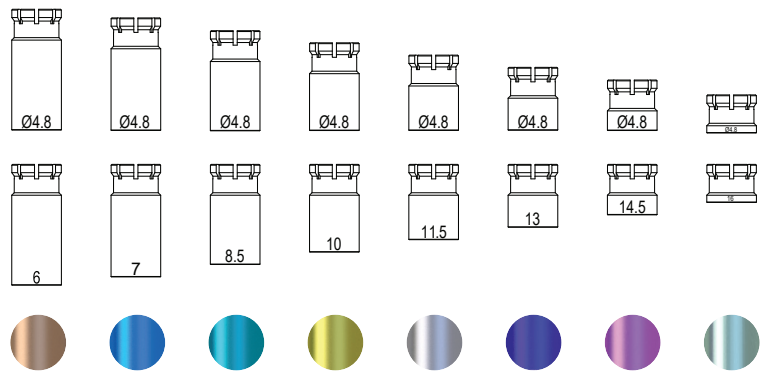**ADVERSE EFFECTS**

Adverse effects will only occur if the choice of instrument or its handling is inadequate, which can lead to unsuccessful of the drilling and even damage to the patient.

**STORAGE CONDITIONS**

The product should be stored in a dry place and in the original packaging at room temperature and not expose to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

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**LIFE CYCLE**

The Medens Easy Drills should be disposed when it loses its functionality.

**SHELF LIFE**

Written on the product label.

Heights	Colors
6,0mm	Copper
7,0mm	Light Blue
8,5mm	Green
10,0mm	Yellow
11,5mm	Lead
13,0mm	Dark Blue
14,5mm	Lilac
16,0mm	Silver

Easy Drills are offered color-coded drill height marking and equivalent drill diameter marking.

\* Illustrative images colors may vary.

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